


CareDry® System: Patient Placement Techniques

Set Suction to Continuous 125 mmHg


FEMALE ANATOMY

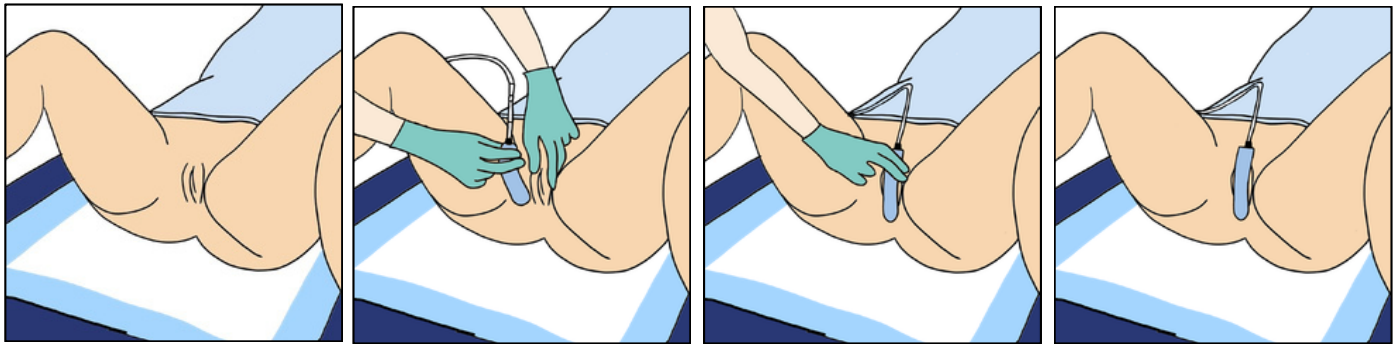

- Separate the patient's legs, gluteus, and labia.
 - Tuck the device vertically, sponge down, between anatomy.
- (Note: If needed, the CareDry® device can be bent to conform to the patient's anatomy.)


- Gently close the legs and place the tubing and connected side of the CareDry® device in a comfortable location where it is unlikely to become dislodged or crushed.
- Device is now ready for use.

EMACIATED -STATIONARY

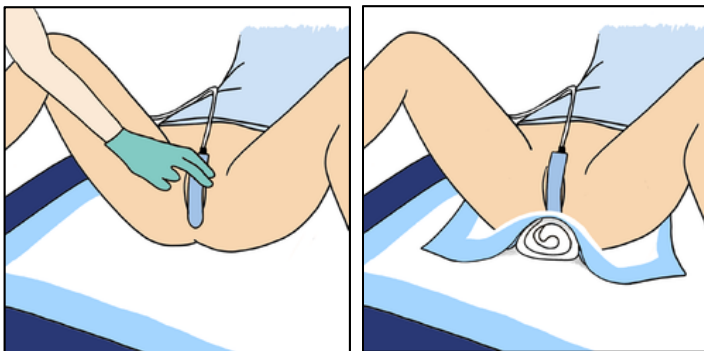


- Once placed based on the instructions above, fold a washcloth to the size of your closed fist and put it under a chuxs pad.
- Place the CareDry® device and edge the washcloth against chuxs to support the CareDry® sponge.

EMACIATED -RESTLESS

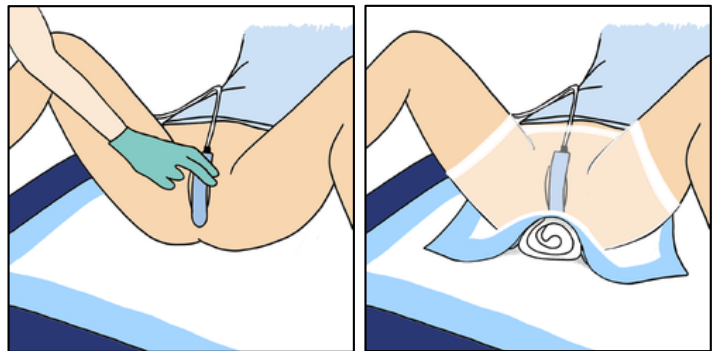

- Place CareDry® device based on instructions above and under "Emaciated -Stationary."
- Then use mesh panties to keep in position -You may use paper tape or stat lock to keep suction extension tubing in place.

OBESE ANATOMY

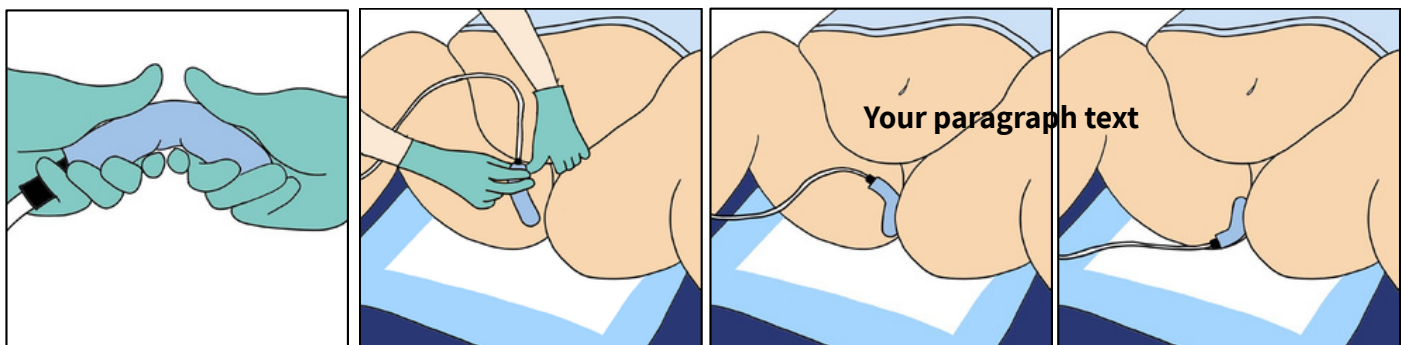

- Ensure the patient's flesh is supported for proper placement.
- Form the CareDry® device into an "S" shape to allow full engagement with the target urine evacuation zone.

- Place the CareDry® device with the connected tubing end either up or down.
- Ensure tubing is directed away from the patient's flesh to travel down the leg and over the bedside.



In
Service
Video

This guide provides suggestions as to how to best place the CareDry® in patients with different anatomies. Refer to the complete Instructions for use included with the product 7660 CareDry® system IFU 34393 for proper use of the device including indications, contraindications, and warnings.



Connecting Healthcare Buyers with Suppliers

CareDry® is a registered trademark of Shortridge Innovations, LLC.